Dental Zirconia

Esthetic Colorant

For KATANA™ Zirconia

ENGLISH INSTRUCTIONS FOR USE

I. Introduction

This IFU is for Esthetic Colorant. Esthetic Colorant is used for coloring pre-sintered zirconia structures and has 12 shade variations: (A plus, B plus, C plus, D plus, BROWN, ORANGE, BLUE, GRAY, VIOLET, WHITE, OPAQUE, and PINK).

II. Indications for Use

Esthetic Colorant is used for coloring pre-sintered zirconia structures made from KATANA Zirconia for the fabrication of all-ceramic restorations.

III. Composition

Water, Propane-1,2-diol, inorganic pigments, etc.

IV. Directions for Use

- To achieve consistent coloring results, any dust or grease must be removed from the surface of the pre-sintered zirconia before applying the liquid.
- Select the color shade.
- Make sure the lid is closed and shake the bottle well prior to use.
- Apply GRAY and BLUE to reproduce the enamel color of the occlusal surface and cusp.
- Apply ORANGE to reproduce mamelon color.
- Apply A plus, BROWN, etc to the main groove to reproduce the dentin color.
- Dry at 80-200°C (176-392°F) for at least 30 minutes.
- Sintering is performed according to the instruction manual of each zirconia.

【Precautions in use】

- Esthetic Colorant is only available with brush painting technique. Apply using a commercially available brush or brush pen. The dipping technique is not recommended.
- Do not mix each shade of Esthetic Colorant, but it is possible to apply each shade in layers to pre-sintered zirconia.

V. Remarks on Handling

Contraindications:

- If the patient is hypersensitive to zirconia or any other components, this product must not be used.

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Warning:

- If the patient or the dental professional demonstrates a hypersensitivity reaction, such as rash, dermatitis etc., discontinue use of the product and seek medical attention immediately.

Caution:

- Do not touch the skin or eyes when using.
- If the product touches human body tissues, take the following actions:
 - <If the product gets in the eye>
 - Immediately wash the eye with plenty of water and consult a physician.
 - <If the product comes in contact with the skin>
 - Immediately rinse with plenty of soapy water.
- Do not mix this liquid with other products.
- Precautions for CMR substance (in accordance with CLP Regulation: Regulation (EC) No 1272/2008)
 - Read precautions in this instruction for use.
 - Do not handle until all safety precautions have been read and understood.
 - Use personal protective equipment as required.
 - If exposed or concerned: Get medical advice/attention.
 - Store locked up.
 - Dispose of contents/container in accordance with local/regional/national/ international regulation.

Storage:

- Store in a cool and dry place. Keep away from direct sunlight.
- The product should be stored at 10-30°C (50-86°F).
- After opening this product, be sure to keep the bottle lid tightly closed.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

If a serious accident attributable to this product occurs, report it to the manufacturer’s authorized representative shown below and the regulatory authorities of the country in which the user patient resides.

Refills do not include instructions for use (IFU). We provide the IFU in electronic format on the dedicated eIFU website.

"KATANA" is a trademarks of NORITAKE CO., LIMITED.

MD	Medical Device
------------------------	----------------

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