CARIES DETECTOR

ENGLISH INSTRUCTIONS FOR USE

I. INTRODUCTION

CARIES DETECTOR clearly differentiates clinically the two layers of carious dentin by staining the outer carious dentin scarlet red yet not staining either the inner carious dentin or normal dentin. The distinct staining of only the outer, infected carious layer provides a visible, clinically precise guide for caries removal.

II. INDICATIONS

CARIES DETECTOR is indicated for the following uses:

- Detecting carious dentine for cavity preparation

III. PRECAUTIONS

1. *Safety precautions*

- Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using product, cover the patient eyes with a towel to protect the patient's eyes from splashing material.
- If the product comes in contact with the human body, take the following actions:
 - <If the product gets in the eye>
 - Immediately wash the eye with copious amounts of water and consult a physician.
 - <If the product comes in contact with the skin>
 - Immediately wipe it off with a cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
 - <If the product comes in contact with the mucous membrane>
 - The product might stay between other medical devices (e.g.cotton rolls) and the mucous membrane. Immediately wipe it off with cotton pledget moistened with alcohol or gauze and wash with copious amounts of water.
- Use caution when using the product to prevent swallowing.
- Do not use the same disposable brush for different patients to prevent cross contamination. Discard the brush after use.
- Avoid direct contact with the skin and / or soft tissue to prevent hypersensitivity.
 - Wear gloves or take appropriate precautions when using the product.
- Do not apply the product directly from the bottle to carious cavity.

2. *Handling and manipulation precautions*

- The use of this product is restricted to a licensed dental professional.
- Provide moisture and contamination control with the use of a rubber dam.
- Do not mix the product with any other dental materials.
- Do not use the product for any purposes other than those specified in INDICATIONS.

3. *Storage precautions*

- The product must be used by the expiration date indicated on the package.
- The product must be stored at 2-25°C/ 36-77°F when not in use.
- Keep away from extreme heat or direct sunlight.
- The product must be stored in proper places where only dental practitioners can access it.

IV. COMPONENTS

Please see the outside of the package for contents and quantity.

1. *Components*

CARIES DETECTOR

2. *Ingredients*

- Propane-1,2-diol
- Dye

V. CLINICAL PROCEDURES

- Isolate carious tooth with a rubber dam prior to application of the product.
- After washing and drying the carious tooth, place one drop of the product on a disposable brush and apply it to the cavity.
- After 10 seconds, wash it with water.
- Remove the exposed outer carious dentin which is stained red.
- Repeat detection and reduction a few more times to insure complete removal of the outer carious dentin.

[NOTE]

- Recommended method for removal of carious dentin:
 - Only the non-sensitive stainable tissue should be removed, carefully avoiding cutting the sensitive unstainable tissue.
 - A low-speed rotary instrument should be used for carious dentin removal since the friction heat generated by a high-speed tool may conduct pain sensation even before reaching the sensitive unstained tissue.
- Cavity preparation procedure using the product:
 - The product can be used in two ways:
 - The product is applied to the cavity area before opening the cavity. This procedure is suggested for a shallow open cavity because earlier disclosure of undermined free enamel may facilitate opening the cavity.
 - The product is used to disclose the residual outer carious dentin after opening a cavity and removing heavily discolored and highly softened dentin. This procedure is suggested for a deep cavity with a large amount of carious dentin for expedience. Careless gross reduction before application of the product, however, may cause pain from unintentional over-reduction.

[CAUTION]

Federal (U.S.A.) law restricts this device to sale by or on the order of a licensed dentist.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

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