

Noritabo

CLEARFIL DC Activator



Make Your CLEARFIL[™] Adhesive A Truly Universal Dual-Cure System Today!

- Low Bond Thickness For Indirect Restorations
- Compatible with Self and Dual-Cure Cements and Core Build-up Composites
- TOUCH-CURE Affect
- Inherited Technology of 35-Year, Proven, Long-Term Track Record with Kuraray Self and Dual-Cure Adhesives

KURARAY AMERICA, INC. 33 Maiden Ln., Suite 600D, New York, NY 10038 800 879-1676 | kuraraydental.com ©2014 Kuraray America, Inc.

MAKES 3 CLEARFIL™ ADHESIVES DUAL-CURE.

CLEARFIL DC Activator activates the dual-curing mechanism of CLEARFIL BONDING SYSTEMS. It's indicated for use with CLEARFIL SE BOND 2, CLEARFIL SE Protect & CLEARFIL Universal Bond.

When mixed with the bond from these systems they become dualcure, truly universal adhesives with low film-thickness & high selfcure bond strengths. This unique combination provides a simple, universal solution for all of your indirect & core build-up restorations.



CLEARFIL DC CORE Plus, the CLEARFIL DC Activator is not required.

For detailed information on the products listed above, please refer to their specific IFU.

GOOD BOND STRENGTHS IN SELF-CURE MODE

Since 1983, with the introduction of **PANAVIA EX** self-cure resin cement, Kuraray adhesives have always included unique, advanced catalyst system for selfcuring. **CLEARFIL DC Activator** contains a modern, advanced accelerator that provides good bond strengths in the self-cure mode when compared to other adhesives on the market today. This advanced chemistry gives you the option to use selfcure or light-cure modes with your indirect restorations.

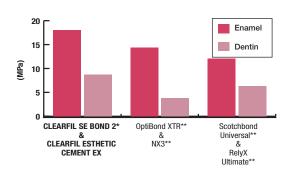



Fig.1 Tensile bond strength to tooth structure with resin cements without light-cure.

Source: Kuraray Noritake Dental Inc.

*CLEARFIL SE BOND 2 BOND was mixed with CLEARFIL DC Activator. ** Not a trademark of Kuraray Co., Ltd

LOW FILM THICKNESS

CLEARFIL DC Activator contains a solvent which reduces the film thickness of the CLEARFIL bonding agents to approximately 3-5 µm. This creates an ideal adhesive layer for indirect restorations.

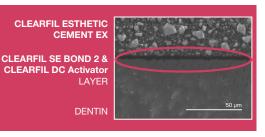


Source: Kuraray Noritake Dental Inc.

TOUCH-CURE

When curing, adhesive resins & composites shrink. With the **CLEARFIL DC Activator**

& CLEARFIL bonding agent combinations, Kuraray's resin cement or core build-up material is activated when it touches the mixture. This TOUCH-CURE affect reverses the shrinkage and starts curing the material at the tooth interface first, which provides an exceptional seal & long-term bond.

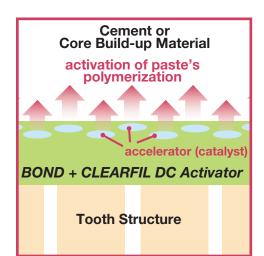



Warranty: Kuraray Noritake Dental Inc. will replace any product that is proved to be defective. Neither Kuraray Noritake Dental Inc. nor the distributor accepts liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user should determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therein. Kuraray Noritake Dental Inc. products are available only through Kuraray Authorized dealers.

"CLEARFIL", "CLEARFIL SE" and "PANAVIA" are trademarks of KURARAY Co., Ltd.

Manufactured by Kuraray Noritake Dental Inc. 1621 Sakazu, Kurashiki, Okayama 710-0801, Japan