K-ETCHANT Syringe

ENGLISH INSTRUCTIONS FOR USE

I. INTRODUCTION

K-ETCHANT Syringe is an etching gel that consists of 35% phosphoric acid aqueous solution and colloidal silica. It allows for precise and selective placement thanks to its flowable but not runny consistency.

II. INDICATIONS

- Etching of enamel and dentin
- Cleaning of dental restorative materials

III. CONTRAINDICATIONS

Patients with a history of hypersensitivity to this product

IV. POSSIBLE SIDE EFFECTS

K-ETCHANT Syringe may cause inflammation or erosion with its irritancy.

V. PRECAUTIONS

1. Safety precautions

- The product may cause inflammation or erosion with its irritancy. Use caution to prevent the product from coming in contact with the skin or getting into the eye. Before using the product, cover the patient's eyes with a towel to protect them in the event of splashing material.
- If the product comes in contact with human body tissues, take the following actions:
<If the product gets in the eye>
Immediately wash the eye with copious amounts of water and consult a physician.
<If the product comes in contact with the skin or the oral mucosa>
Immediately rinse with copious amounts of water.
- This product contains substances that may cause allergic reactions. Avoid use of the product in patients with known allergies to any components in the product.
- If the patient demonstrates a hypersensitivity reaction, such as rash, eczema, features of inflammation, ulcer, swelling, itching, or numbness, discontinue use of the product and seek medical attention.
- Wear gloves or take other appropriate protective measures to prevent the occurrence of inflammation, erosion or hypersensitivity that may result from contact with the components in the product.
- Use caution to prevent the patient from accidentally swallowing the product.
- The needle tip is single-use only. Do not reuse it to prevent cross-contamination. Discard it after use.
- If the instruments associated with this product are damaged, use caution and protect yourself; immediately discontinue use.
- If the product adheres to clothing, wash it off with water.

2. Handling and manipulation precautions

- The product must not be used for any purposes other than specified in [II. INDICATIONS].
- The use of this product is restricted to licensed dental professionals.
- Use a rubber dam to control moisture, if necessary.
- After each use, remove the needle tip from the syringe and recap the syringe immediately and tightly.
- Do not mix the product with other materials.

3. Storage precautions

- The product must be used by the expiration date indicated on the package.
- The product must be stored at 2 - 25°C / 36 - 77°F when not in use.
- The product must be kept away from extreme heat or direct sunlight.
- The product must be stored in a proper place where only dental practitioners can access.

VI. COMPONENTS

Please see the outside of the package for contents and quantity.

- Etching agent
Principal ingredients:
 - Phosphoric acid
 - Water
 - Colloidal Silica
 - Pigment
- Accessory
 - Needle tip (E)

VII. CLINICAL PROCEDURE

1. Syringe preparation

- 1-1. Unscrew the cap of the syringe, and attach the needle tip to the syringe by rotating clockwise. Cover the whole syringe with a disposable barrier (e.g. a poly bag) to prevent contamination from saliva or blood. Disinfect the syringe by wiping it with an absorbent gauze pad moistened with alcohol both before covering and after use.
- 1-2. Dispense a small amount of the gel onto a pad or gauze, and make sure the gel is dispensed adequately.

2. Pretreatment of the adherent area

Pretreat the adherend area according to the instructions for use of the adhesive system being used together.

3. Application of the gel

Dispense the gel slowly and apply it to the adherend surface. Refer to the instructions for use of the adhesive system being used together to confirm the appropriate etching time. The standard etching time of this product is 5-10 seconds.

4. Rinsing and Dry

After the predetermined time, rinse thoroughly and dry in the usual manner.

5. Application of dental adhesive

Proceed with the work using the adhesive system according to the specific instructions for use of its manufacturer.

[CAUTION]

- Regarding the etching conditions, follow the instructions for use of the adhesive system being used together.
- Avoid any contamination to the etched tooth surface with saliva, etc. If contamination occurs, repeat the process of etching, rinsing, and drying.

[CAUTION]

Federal (U.S.A.) law restricts this device to sale by or on the order of a licensed dentist.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

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La legislación federal (U.S.A.) limita este dispositivo a la venta por o bajo pedido de un dentista autorizado.