


CLEARFIL™ Universal Bond Quick

Quick without Compromise

- No Waiting Time with Real Clinical Benefits
- Less Technique Sensitive
- Less Risk of Contamination




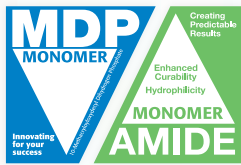

No Waiting Time

The new original MDP & Amide chemistry provides a unique RAPID BOND TECHNOLOGY. Unlike other universal adhesives, there is no need to wait for it to penetrate the dentin before you proceed.

The new **Amide** monomer rapidly permeates dentin & enamel eliminating waiting time.

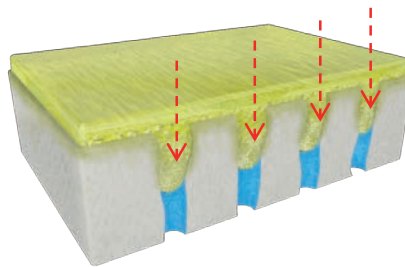

The original **MDP** monomer creates a strong chemical bond to hydroxyapatite. MDP has a proven excellence in Adhesion for over 30 years.

RAPID BOND TECHNOLOGY offers consistent, predictable results with all etching procedures & virtually any application time.




REAL CLINICAL BENEFITS:

- Less Technique Sensitivity
- Less Risk of Contamination
- More Efficiency

Quick without Compromise: Excellent Bond Strength

Self-Etch Shear Dentin Bond Strengths (Notched Edge, MPa)

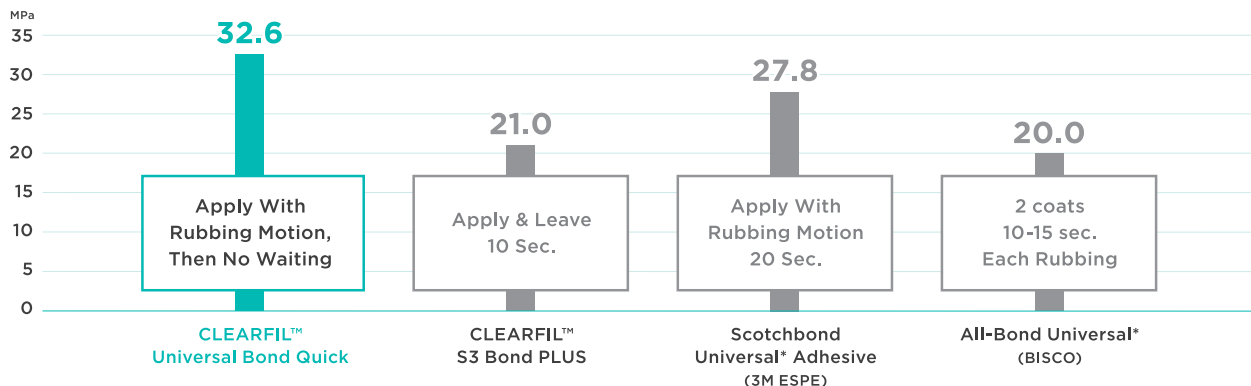

*Not a trademark of Kuraray Co., Ltd Data source: Kuraray Noritake Dental Inc.

A Truly Universal Multi-purpose Adhesive

All Etching Techniques



Self-etch
no phosphoric acid

Selective-etch
phosphoric acid on enamel

Total-etch
phosphoric acid on Enamel and dentin

All Clinical Indications



Direct Restoration

Post Cementation/ Core Build-Up

Cementation Indirect Restoration

Intraoral Repair

Virtually no Post-Operative Sensitivity

Low film thickness

Allowing esthetic and good fit of indirect restorations

Best Results in Gordon J. Christensen Clinicians Report

“CLEARFIL™ Universal Bond Quick had best overall results, most notably for its superior bond strength values and best ease of application”

Vol. 11, Issue 5, May 2018

Improved Bottle Delivery

One hand for opening, dispensing and closing

Unique Nozzle for drop control

Clean after use no residual build-up

Minimal waste

Compact



More Features Than Leading Universal and Total-Etch Adhesives

Universal Adhesives

	CLEARFIL™ Universal Bond Quick	Scotchbond Universal Adhesive* (3M ESPE)	ALL-BOND Universal* (BISCO)
Fluoride Release	Yes	No	No
Bonds Zirconia	Yes	Yes	Yes
Contains Silane (Bonds e.max* & Porcelain)	Yes	Yes	No
Economic Bottle Packaging	Yes (3 Bottles)	No	No

Total-Etch Adhesives

	CLEARFIL™ Universal Bond Quick	Optibond Solo Plus* (KERR)	Prime & Bond® NT* (Dentsply Sirona)
Indicated for All Etching Techniques	Yes	No	No
May Be Used as Self-Cure Adhesive	Yes ¹	No	Yes**
Bonds Zirconia	Yes	No	No
Bonds Lithium Disilicate	Yes	No	No

*All information from mfg. Instructions For Use. Not a trademark of Kuraray Co., Ltd. **Must purchase a separate dual-cure activator
1. Indicated for use in self-cure mode with PANAVIA SA Cement Plus.

Basic Procedure



Direct Restorations using Light-Cured Composite Resin



Self-etch mode
Move to next step

Selective-etch mode 10 seconds
Rinse & Dry

Total-etch mode 10 seconds
Rinse & Dry



Apply with a rubbing motion



(No waiting time after application)



Dry
5 seconds
use a vacuum aspirator to prevent Bond from scattering




Light-cure
5 seconds(High performance LED)

Indirect Restorations Cementation of Indirect Restorations with PANAVIA™ SA Cement Plus to Zirconia Crown



Clean and dry
Clean and dry the tooth surface, and trial fit the restoration

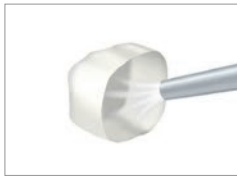

Blast
If necessary, Blast with alumina powder (30-50µm, 0.1-0.4MPa / 15-58PSI / 1.4kgf / cm²) then ultrasonic clean and dry



Tooth Pretreatment
a) Self-etching (move to next step)
b) Selective-etching
c) Total-etching



Apply
Apply CLEARFIL™ Universal Bond Quick with a rubbing motion and dry by blowing mild air



Cementation
Cementation using PANAVIA™ SA Cement Plus according to the manufacturer's instructions



Ordering Information



**#3571KA
Bottle Standard Kit**
1-Bond (5ml)
1-K-ETCHANT Syringe (3ml)
& 20 Needle Tips
1-Dispensing Dish
1-Light-Blocking Plate
50-Applicator Brushes (Fine Silver)



**#3572KA
Bottle Refill**
1-Bond (5ml)



**#3574KA
Bottle Value Pack**
3-Bonds (5ml each)



**#3577KA
Unit Dose Standard Pack**
50 Unit-Dose
1-K-Etchant Syringe (3ml)
& 20 Needle Tips
50-Applicator Brushes (Fine Silver)

**#3578KA
Unit Dose Value Pack**
100 Unit-Dose
1-K-Etchant Syringe (3ml)
& 20 Needle Tips
100-Applicator Brushes (Fine Silver)

YOUR CONTACT

Kuraray America, Inc.

32 Old Slip Phone (800) 879-1676
Floor 7 Fax (888) 700-5200
New York, NY 10005 E-mail info@kuraraydental.com
Website www.kuraraydental.com

- Before using this product, be sure to read the Instructions for Use supplied with the product.
- The specifications and appearance of the product are subject to change without notice.
- Printed color can be slightly different from actual color.

"PANAVIA" and "CLEARFIL" are trademarks of Kuraray Co., Ltd.



Kuraray Noritake Dental Inc.

Ote Center Bldg.1-1-3, Otemachi, Chiyoda-Ku, Tokyo 100-0004

R& Only



CE 0120