

## CLEARFIL MAJESTY ES-2 Universal Product FAQ

**Q: What is Clearfil Majesty ES-2 Universal?**

A: *CLEARFIL MAJESTY ES-2 Universal is the latest addition to the MAJESTY “ ES-2 series. Like the Classic and Premium series, it is a light-cure, radiopaque restorative material which provides accurate color matching, high polishability and excellent physical properties. It is recommended for use in the posterior region of the mouth. Formulated with optimal viscosity, it assures easy handling and placement. The composite is available in both traditional syringes or Pre-loaded tips (PLTs). What makes this Universal truly special is that one shade covers all shades of the VITA classic A1-D3 shade guide. The Universal shade is recommended for all Class I, II and V restorations.*

**Q: How does CLEARFIL MAJESTY™ ES-2 Universal series differ from the Classic and Premium series?**

A: *CLEARFIL MAJESTY™ ES-2 Universal is a simplified system that covers all the shades of the VITA classical A1-D3 shade guide. The mechanical properties are identical to those of CLEARFIL MAJESTY™ ES-2 PREMIUM and CLASSIC.*

**Q: How many shades are available and how will they be used?**

A: *There is a single shade of CLEARFIL MAJESTY™ ES-2 Universal available. A single shade “U- Universal” will cover all the posterior restorations (Classes I, II, V).*

**Q: How does the Universal series cover all the colors of the VITA classical A1-D3 shade standard with only one shade?**

A: *The improved light diffusion of CLEARFIL MAJESTY™ ES-2 Universal offers superior blending to tooth structure. Adjustments have been made regarding the translucency/ light diffusion balance, allowing for a further improved color adaptation to the surrounding tooth structure.*

**Q: When do you recommend the use of the different ES-2 series?**

A: *Universal, Classic and Premium series can be used for all cases.*

- *Universal shade is recommended when looking for a simplified and easy-to use system that will work for the vast majority of your daily cases.*
- *Classic shades offer more shade options for specific cases.*
- *Premium shades are recommended for those cases with the highest aesthetic demands or for large cavities in the anterior area. The balance of opacity and translucency can be adjusted by using both enamel and dentin-specific shades.*

**Q: Does CLEARFIL MAJESTY™ ES-2 Universal have a long working time?**

A: *Yes. CLEARFIL MAJESTY™ ES-2 Universal keeps the same handling and mechanical properties as the Classic and Premium series. It has a significantly longer working time than competitive products (over 4 minutes under ambient light). Having a longer working time is very beneficial when sculpting the anatomical aspects into the restoration.*

**Q: What is the filler content of CLEARFIL MAJESTY™ ES-2 Universal?**

**A:** *The filler content of CLEARFIL MAJESTY™ ES-2 Universal amounts to 78 wt% (66 vol%).*

**Q: Can CLEARFIL MAJESTY™ ES-2 Universal be easily polished?**

**A:** *The polishability is the same as already established with MAJESTY™ ES-2 series. CLEARFIL MAJESTY™ ES-2 Universal offers easy polishing with a durable gloss.*

**Q: How will CLEARFIL MAJESTY™ ES-2 Universal be supplied?**

**A:** *CLEARFIL MAJESTY™ ES-2 Universal will be supplied in a syringe, as well as PLT delivery (PLT = pre-loaded tip).*

**Q: Do I have to keep the product in the refrigerator?**

**A:** *CLEARFIL MAJESTY™ ES-2 Universal must be stored at 36-77°F when not in use. When the product is stored in the refrigerator, it should be left at room temperature for more than 15 minutes before each use to ensure the desired handling properties.*

**Q: I do not have a shade guide for U. How is shade selection accomplished?**

**A:** *The concept of the Universal is that a shade guide is not needed in order to work in a simplified way. As a reference, use U shade for all posterior cavities.*

**Q: What does ES-2 stand for?**

**A:** *ES-2 represents Esthetic and Simple.*

**Q: What is the shelf life?**

**A:** *The shelf life is the same as the rest of CLEARFIL MAJESTY™ ES-2 series: 48 months from date of manufacture.*

