(TI-012)

Dental Zirconia

KATANA™ Zirconia

ST (Super Translucent) & STML (Super Translucent Multi Layered)

ENGLISH INSTRUCTIONS FOR USE

I. Introduction

KATANA Zirconia has 2 shapes available: Block and Disc. This IFU is for KATANA Zirconia ST (Super Translucent) & STML (Super Translucent Multi Layered). KATANA Zirconia ST & STML are a pre-sintered zirconia disc 98,5 mm in diameter, which contains a plastic ring. This is designed for all milling systems using this generic-type disc. (Please refer to your milling system’s technical instructions for correct machine operation.)
KATANA Zirconia ST has 3 available thicknesses: (14 mm, 18 mm and 22 mm) and has 1 shade variation: (ST10). KATANA Zirconia STML has 3 available thicknesses: (14 mm, 18 mm and 22 mm) and has 5 shade variations: (A1, A2, A3, A3.5, NW).
KATANA Zirconia ST is mono color and KATANA Zirconia STML consists of 4 gradated shade layers. KATANA Zirconia ST & STML are recommended for use in fabricating FCZ (Full Contour Zirconia) restorations or the frameworks.

II. Intended Use

KATANA Zirconia is used for the fabrication of the all-ceramic restorations (frameworks, FCZ crowns, FCZ bridges, inlays, onlays and veneers).

III. Sintering Program

Temperature	Programming Rate	Holding Time			
Room Temp. — 1550°C (2822°F)	10°C/min. (18°F/min.)	—		1550°C (2822°F)	
1550°C (2822°F)	—	2 hrs		Hold for 2 hrs	
1550°C (2822°F) — Room Temp.	-10°C/min. (-18°F/min.)	—	Room Temp.	Room Temp.	

IV. Composition

ZrO₂, Y₂O₃ etc.

V. Type and Class (ISO6872:2008)

Type:II/ Classe:5

VI. Physical Properties

Coefficient of Thermal Expansion (25-500°C (77-932°F)): 9.8(±0,2)×10⁻⁶/K

VII. Directions for Use

- Take the disc from the packaging and confirm that the disc does not have a crack or other damage.
- Place the disc into the milling machine; then begin the milling process following the milling systems technical instructions.
- After milling, remove the restorations from the disc with a diamond bur etc.
- The cutting waste or dust, which is attached to the restorations, can be removed with a gentle air stream.
- Put the restorations into the refractory sagger tray and place them in the sintering furnace.
- Sinter the restorations according to the sintering schedule shown above. (III. Sintering Program).
- After sintering, adjust the restorations with a diamond bur as needed.
- Confirm that the restorations have no cracks.
- 9)-1 FCZ:
 - Glaze baking: Create a high shine surface by polishing, especially on the contact areas then apply the glaze on all surfaces in the usual manner.
 - Hand polishing: Create a high shine surface on the entire restoration by polishing without using the glaze. When the restoration is to be finished without the glaze, select a one shade brighter than the final shade.
- 9)-2 Frameworks: Build-up the dental porcelain (CERABIEN ZR or CZR PRESS LF etc.) on the frameworks following the manufacturer’s technical instructions. Check the coefficient of thermal expansion of the porcelain in the manufacturer’s technical instructions to confirm compatibility.

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VIII. Remarks on Handling

Contraindications:

• If the patient is hypersensitive to zirconia or any other components, this product must not be used.

Caution:

- If the patient or the dental professional demonstrates a hypersensitivity reaction, such as rash, dermatitis etc., discontinue use of the product and seek medical attention immediately.
- When milling the disc or cutting, grinding and polishing the restorations, use an approved dust mask and vacuum with air filter to protect your lungs from inhaling dust.
- When milling the disc or cutting, grinding and polishing the restorations, use safety glasses to prevent the dust from getting into your eyes. If the dust gets into your eyes, immediately rinse with copious amounts of water and consult a physician.
- Do not use for any purposes except for dental restoration. This product is for dental application only.
- Do not touch the items heated by the furnace with your bare hands.
- Do not use this product to make 4+ unit bridges or cantilever bridges.
- This product is particularly translucent; therefore, one should carefully consider its appropriateness for use when masking the underlying surface, such as a metal abutment, or a very dark or discolored tooth preparation.
- The margins should be prepared with a deep chamfer and rounded shoulders, with cutting edges and corners rounded to eliminate sharp preparation corners. The angle of the axial surface should be within the range of 5 to 15 degrees.
- When preparing teeth, avoid the following: deep shoulders, J-margins, knife edges, serrated margins, non-tapered abutments, undercuts, guide grooves, the formation of retentive holes, and sharp corners.
- Keep the following thickness of this product for fabricating prosthetics:

Location & indication	Wall thickness
Anterior crown or bridge	0.8 mm or more
Veneer	0.4 mm or more*
Posterior crown or bridge	1.0 mm or more
Inlay or onlay	1.0 mm or more

* 0,4 mm or more of this product is for full zirconia veneers. Keep thickness 0,8 mm or more, if it is used for combination with the porcelain.

- Use the following cross-sectional areas for connectors when fabricating bridges.

Location & indication	Connector cross section
Anterior 2- or 3-unit bridges	12 mm² or more
Posterior 2- or 3-unit bridges	16 mm² or more

Caution in conjunction with usage:

- Do not use the disc if there is a crack noticed after removing it from the package.
- Dry milling is recommended. If wet milling is used, the translucency of the disc may be reduced.
- If you find a crack in the restorations, do not use.
- The above sintering recommendation is only a guideline; some adjustments may be required depending on each individual furnace.
- When milling the disc, do not bring the milling bar into contact with the plastic ring to prevent detachment of the disc.
- Cut and remove the plastic ring prior to sintering the whole disc.
- Storage:**
 - Store in a cool and dry place. Keep away from direct sunlight.
 - The product should be stored at 10-30°C(50-86°F).
 - Do not remove the disc from its packaging during storage.
 - The disc is fragile, and requires care when handling.
 - The product must be stored in an appropriate place where only dental personnel have access.
 - The product must be used by the expiration date indicated on the package.

[WARRANTY]

Kuraray Noritake Dental Inc. will replace any product that is proven to be defective. Kuraray Noritake Dental Inc. does not accept liability for any loss or damage, direct, consequential or special, arising out of the application or use of or the inability to use these products. Before using, the user shall determine the suitability of the products for the intended use and the user assumes all risk and liability whatsoever in connection therewith.

[NOTE]

*KATANA® is trademarks of NORITAKE CO., LIMITED.

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